



Sonesta Online Technical Training
October 14, 2020

Welcome!

Please

- Please mute yourself when you are not speaking to avoid background noise.
- Feel free to ask questions directly or formulate them in the chat at any time!



Participants

Participants


Name	Company	Country
Andrew Hucman	Endotherapeutics	Australia
Jaanus Joller	Sonotrade	Estonia
Jonathan Becker	Becker Medical	France
Panagiotis Glavas	BIOSENSE	Greece
George Devekos	BIOSENSE	Greece
Jo Jo Cheung	Tronda	Hong Kong
Desmond Wong	Tronda	Hong Kong
Eason Tsang	Tronda	Hong Kong
James Mulcahy	Tekno Surgical / Med Surgical	Ireland
Neil Maher	Tekno Surgical / Med Surgical	Ireland
Stephen McCormack	Tekno Surgical / Med Surgical	Ireland
Eoin Fowler	Tekno Surgical / Med Surgical	Ireland
Iarla Marron	Tekno Surgical / Med Surgical	Ireland
Colette Walsh	Tekno Surgical / Med Surgical	Ireland
Brendan Garry	Tekno Surgical / Med Surgical	Ireland
Fadi El Hajj	Alpha Pro	Lebanon
Alain Hanna	Alpha Pro	Lebanon
Tamara Akl	Alpha Pro	Lebanon
Kharulnisya Bt Inche Muhd Zulfikri	MKS Medic Sdn Bhd	Malaysia
Lee Wai Kien	MKS Medic Sdn Bhd	Malaysia
Mohamad Hafis Fauzan	MKS Medic Sdn Bhd	Malaysia
Teong Heng, Tan	Radiance Medical System	Singapore
Aye Chan Zaw	Radiance Medical System	Singapore
Hyungsoon Park	HiMed Co., Ltd	South Korea
Jackson Tseng	Kitwell/Giantboat	Taiwan
Eason Chen	Kitwell/Giantboat	Taiwan
Jessie Huang	Kitwell/Giantboat	Taiwan
Jean-March Van Hoeck	Global Lithostripsy Service, Sodexo	Thailand
Noppaklow.A	Sodexo Thailand	Thailand

Trainers

Sonesta; Roger Henriksson

Sonesta/Samhall; Tommy Ståhl

Linak; Fredrik Strandahl, Jan Storm

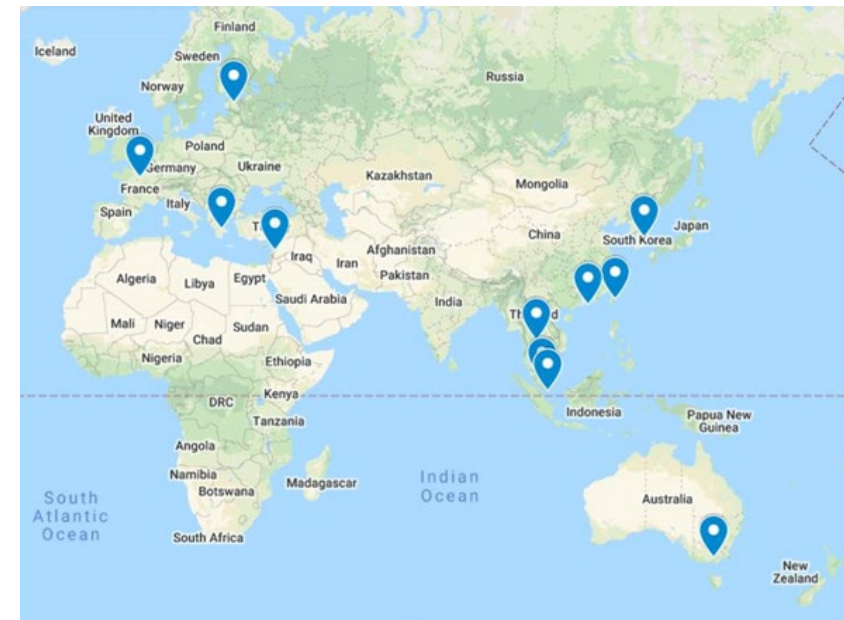

Presentation of the participants	Roger H
Sonesta company presentation	Roger H
Sonesta products	Roger H
Available documentation	Roger H
Mechanical construction of the products	Tommy S
Electrical construction of the products	Linak
Fault finding	Roger H, Tommy S
Common issues	Roger H, Tommy S, Linak
Questions	All

Aim

- To get to know the products better.
- Where to find more information.



WHO ARE WE?

- We manufacture and sell tables for **urology, gynecology and urodynamic diagnostics** focusing on **quality, safety, function and comfort**.
- Market leaders with an installed base of approximately 8.000 chairs and tables.
- We started selling chairs and tables in the beginning of the 1990's. Almost 30 years later, we can still offer an optimal way of performing examinations in a time and cost-efficient way.
- All our products are made in Sweden.



ADDVISE GROUP

- Sonesta was acquired by the ADDvise Group AB (publ) in August 2013.
- ADDvise Group is a growth-oriented group that, through the business areas Lab and Healthcare, offers complete solutions in the form of products and services for the healthcare sector and research facilities.
- Customers of the ADDvise Group are in the private and public sector.
- The company is listed on NASDAQ First North.




GUARANTEED QUALITY

The products are manufactured with the highest quality as to guarantee maximum comfort, functionality and longevity.

- ISO 9001, 14001
- CE Medical Device Directive 93/42EEC Class 1
- UL
- IPX6
- Phtalate-free cushions
- Electric shock protection Type B
- Electromagnetic compatibility EMC
- Isolation and emission in accordance with IEC 60601-1-2



Made in Sweden

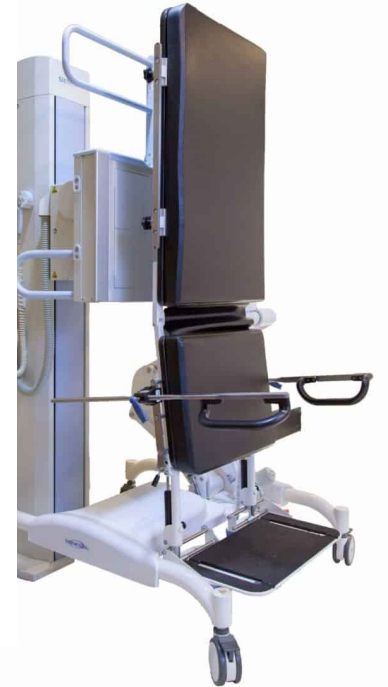

The Sonesta collection

- **S2 & S3**

- Urology
- Urodynamics
- Gynecology
- Pelvic Muscle Rehabilitation
- Anorectal Manometry
- Pelvic Exam
- Cystoscopy
- Ultrasound
- Vasectomy
- Colposcopy
- Biopsy

- **6210**

- Video Urodynamics
- Fluoroscopy Procedures
- as well as:
 - Pelvic Muscle Rehabilitation
 - Anorectal Manometry
 - Pelvic Exam
 - Cystoscopy
 - Ultrasound
 - Vasectomy
 - Colposcopy
 - Biopsy
 - Colonoscopy
 - Gynecologic Procedure

SST

- General x-ray
- Stitching images
- Trauma clinics

<https://www.sonestamedical.se/>

<https://www.sonestamedical.se/manuals/>

<https://www.sonestamedical.se/quality/>

Quality and Environment

Sonesta is certified according to the international standard ISO 14001. Sonesta applies the

Climate change is a real threat to our world, and we have a big role to play in reducing our impact on the environment. For example, we can always choose a more environmentally friendly paper at the office, and only make well-thought-out decisions.

We fully CO2 compensate our travels by charitable environmental organisation wi

Sonesta User Manuals

You can download our User Manuals here. You can also find our User Manuals in other languages.

English

SONESTA S2 & S3 USER MANUAL

SONESTA 6210 USER MANUAL

SONESTA SST USER MANUAL

French

SONESTA S2 & S3 NOTICE D'UTILISATION


SONESTA 6210 NOTICE D'UTILISATION

Spanish


[+ SONESTA S2 & S3 MANUAL DE USUARIO](#)




S-series (NGS)

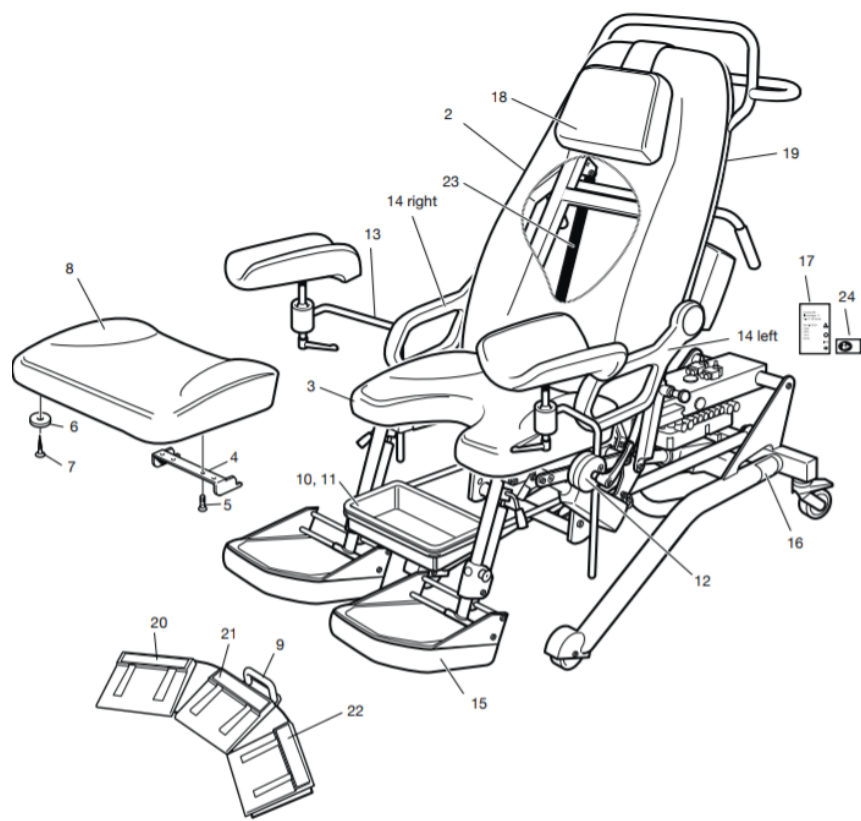


6210 / SST

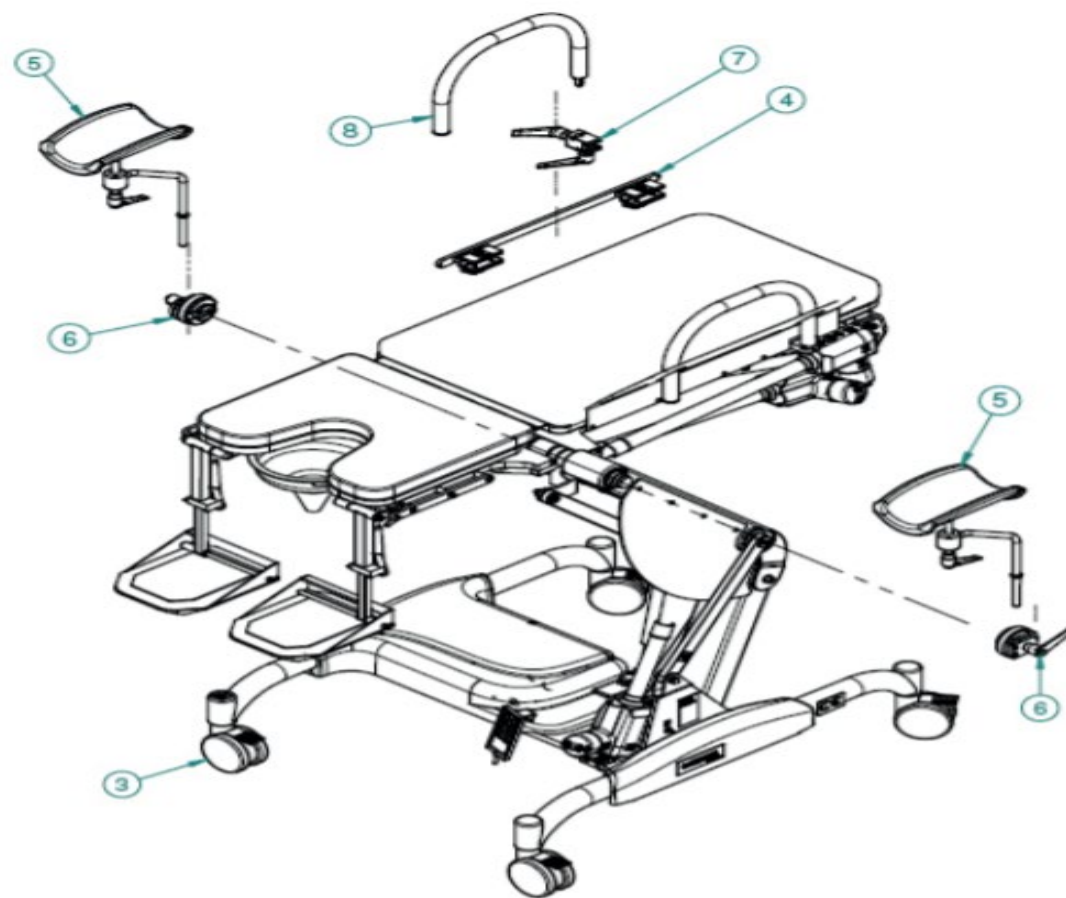


S-series (NGS)

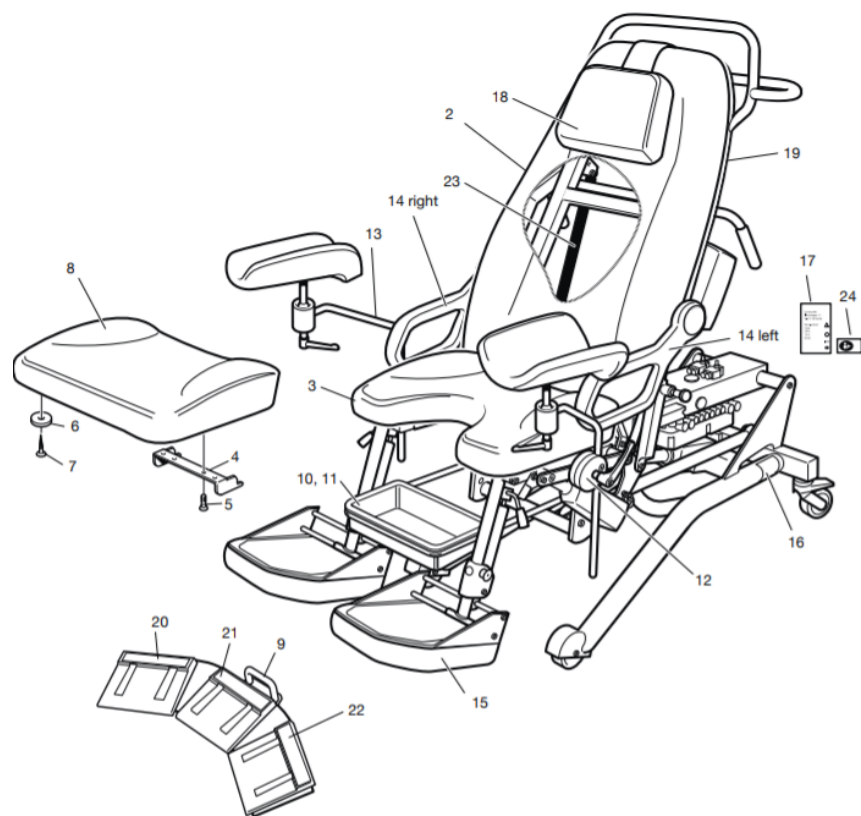


6210 / SST

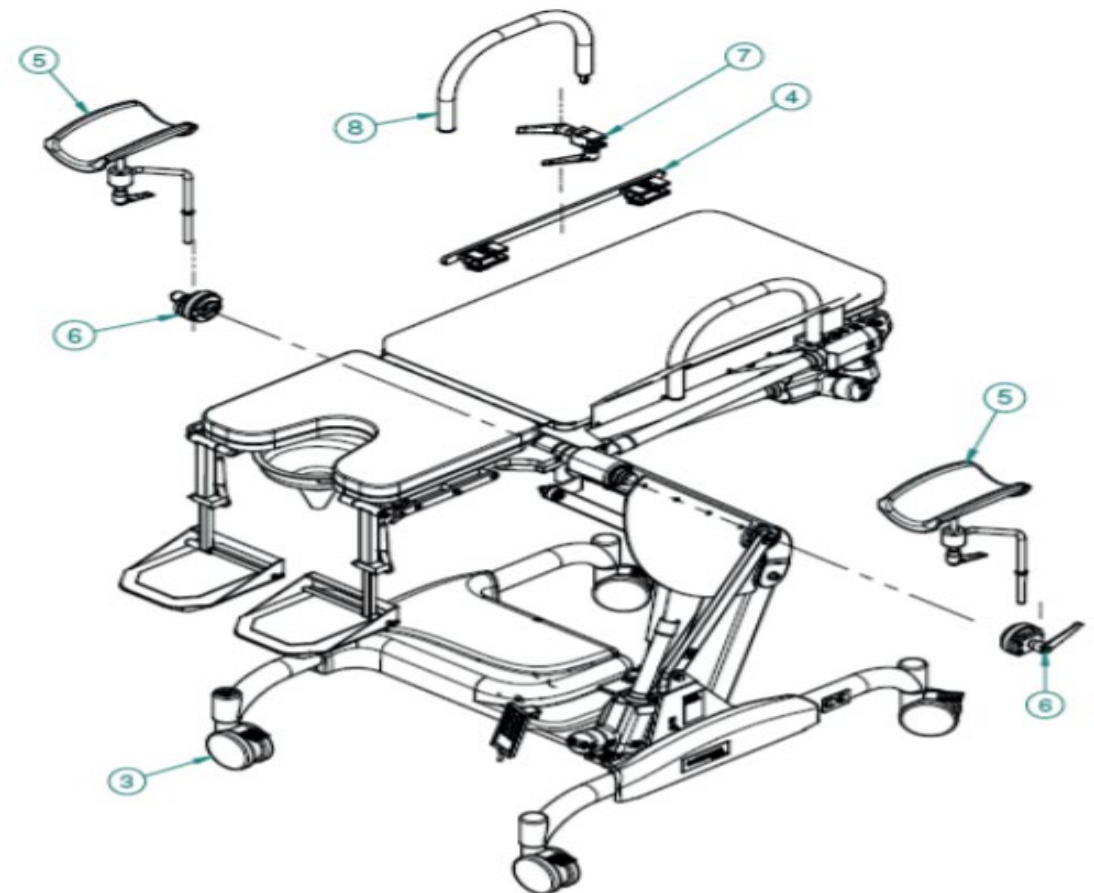


S-series (NGS)

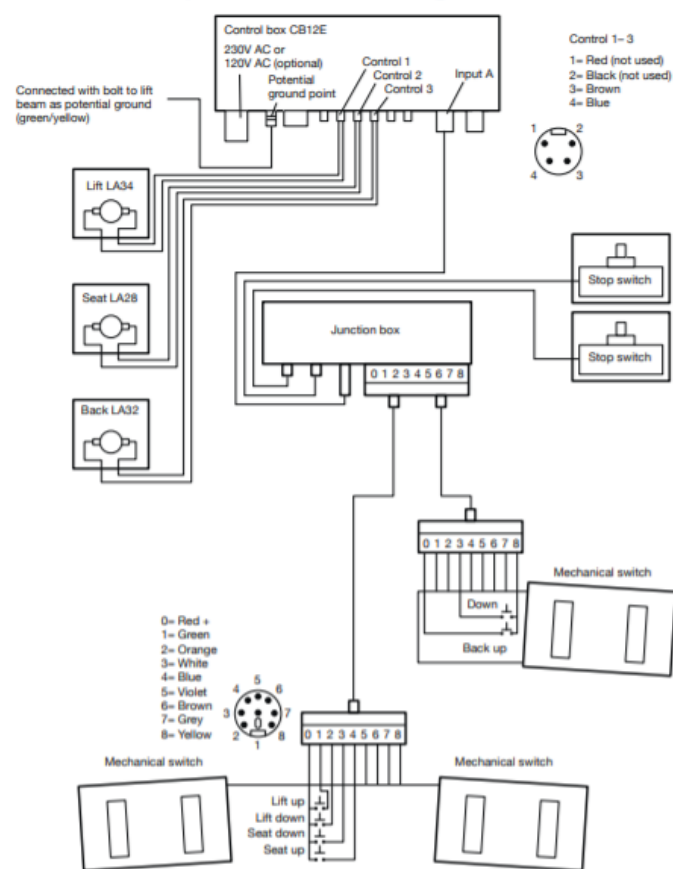


6210 / SST

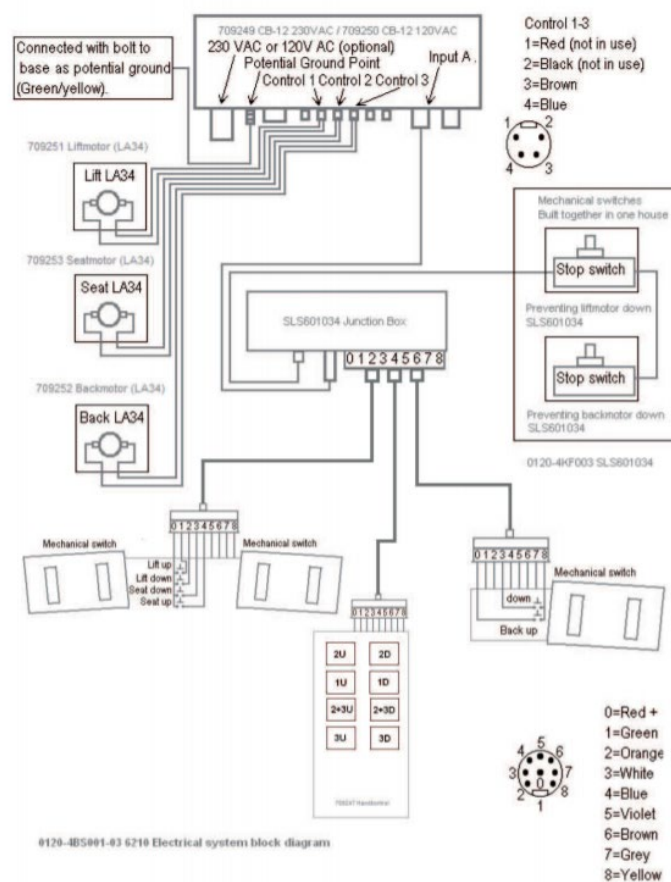

S-series (NGS)

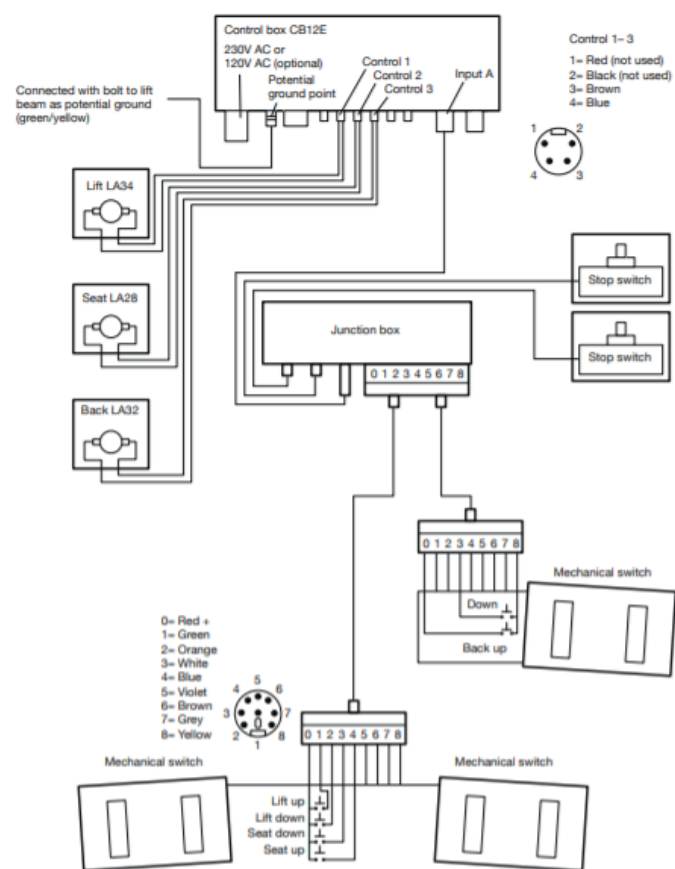

6210 / SST

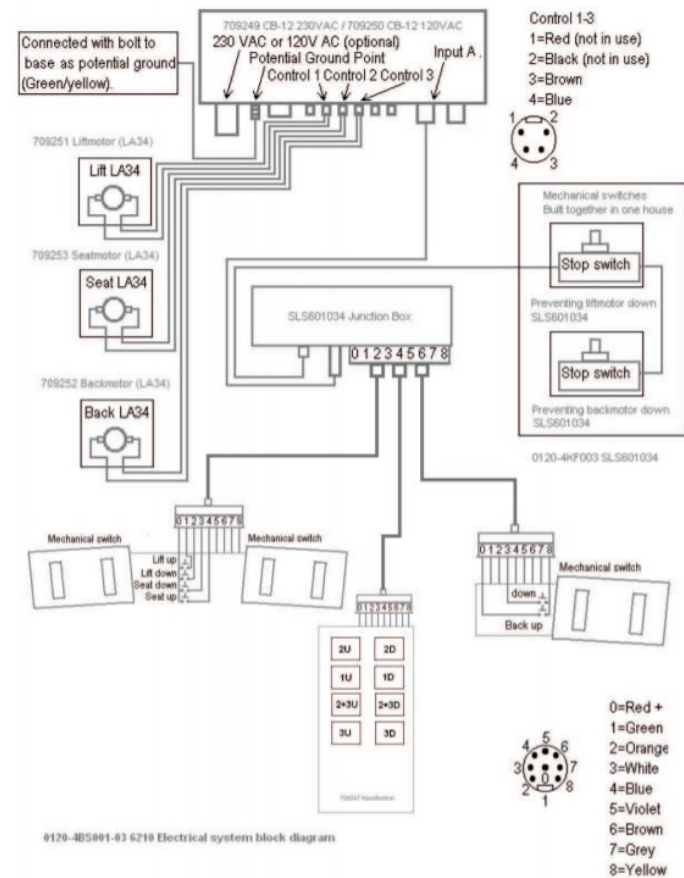



www.sonestamedical.se/serviceinstructions/

Sonesta S-series Technical training & Fault finding




www.sonestamedical.se/serviceinstructions/


Sonesta 6210

Technical training & Fault finding



- Footrest

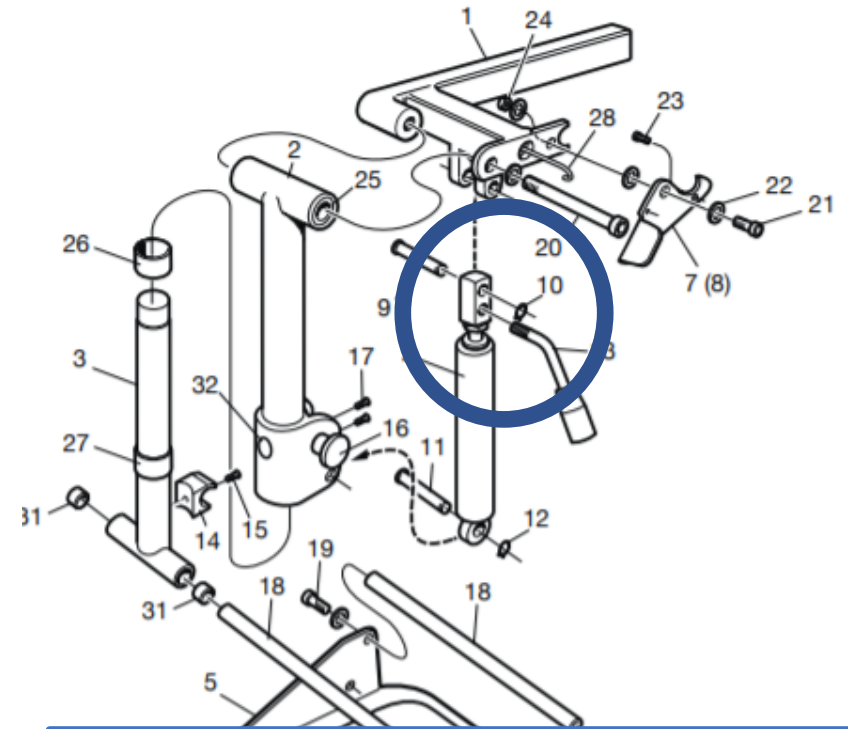

Claim report - Warranty

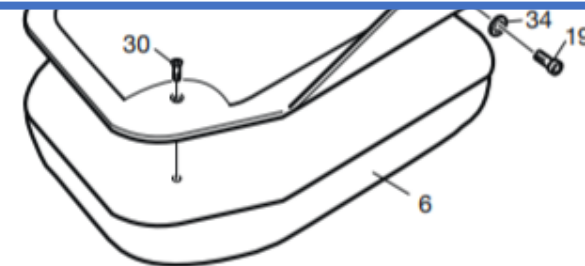



- Emergency stop box (6210, SST)

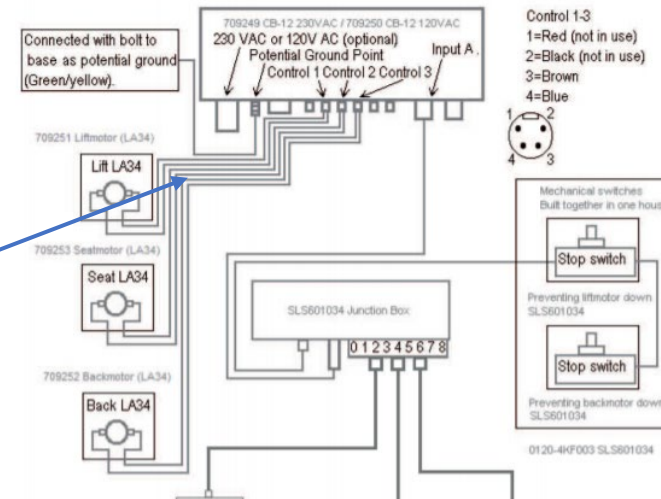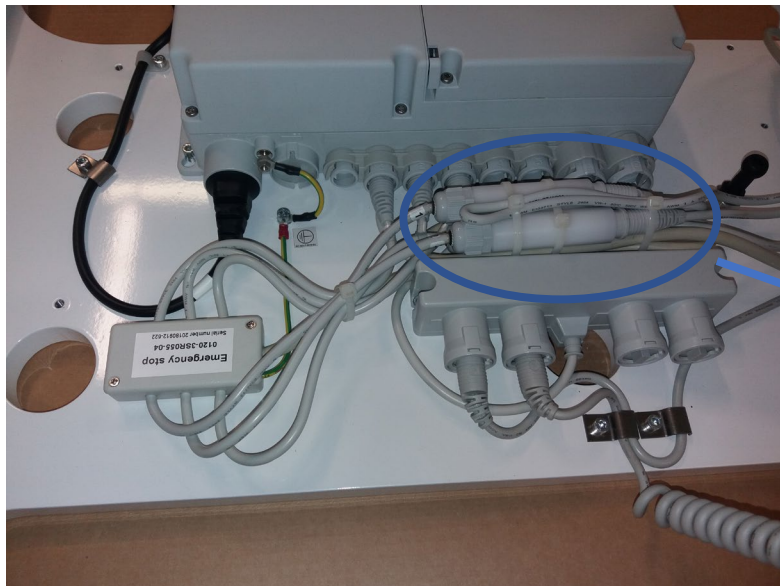

1. Correct any loose contact
2. Replace



- Leg rests

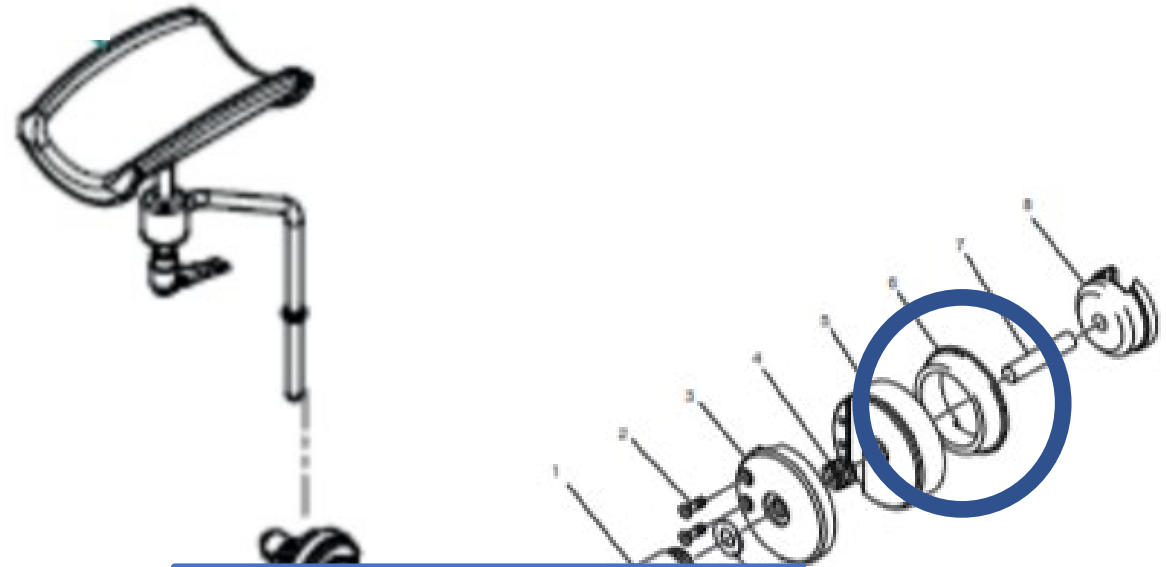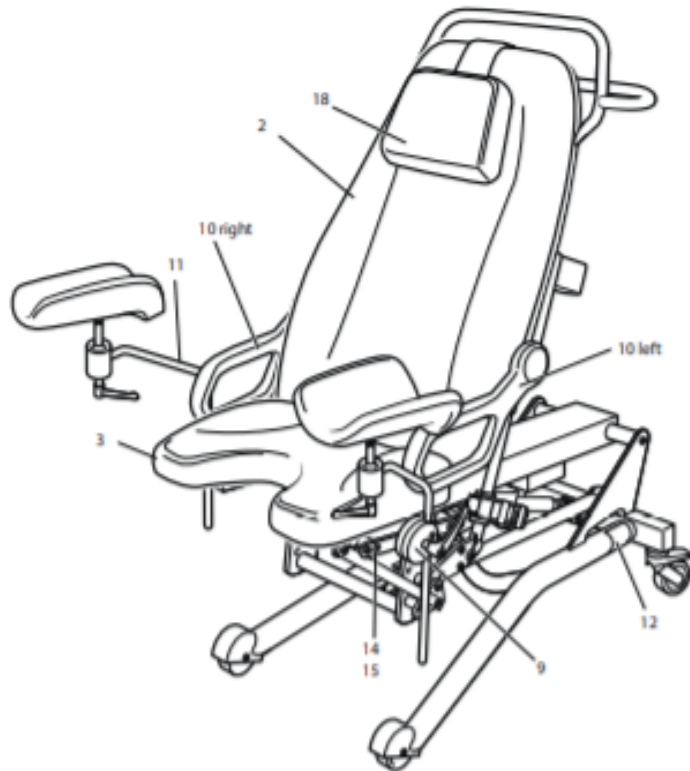

1. Teflon oil
2. Replace

- Corrosion



Extra zinc coating since fall 2018

Cleaning

The unit needs to be unplugged from the wall power before any disinfection.

Wipe all surfaces after each patient use.

Please note that it is recommended to remove the seat cushion after each use and to wipe the surfaces below the seat cushion as well as the back of the seat cushion.

Clean the chair with a damp cloth and ordinary disinfectants. Follow established protocols for cleaning body fluids from the chair surfaces.

Use any of the below listed recommended disinfectants on all surfaces. If they are not available, use any disinfectant with 1:10 dilution of bleach (5.25% sodium hypochlorite).

Recommended disinfectants:

PDI Sani-Cloth SANI-CLOTH® Bleach Wipe: Germicidal Disposable wipe.

Poor maintenance



WARNING: Improper, or lack of cleaning after each use of the chair will cause damage to the appearance and performance.



WARNING: Iodophor type disinfectants (betadine, for example) may stain the cushions.



WARNING: Prolonged contact with bleach solutions with improper dilution ratios (for) in any cleaning agent may cause damage to both the appearance and performance.

- Electronics

Foot/hand control unit and cord has cuts or dent



Replace



More info and Contacts

- www.sonestamedical.se
- www.sonestamedical.se/serviceinstructions/
- complaint@sonestamedical.se
- info@sonestamedical.se
- roger.henriksson@sonestamedical.se

Sonesta User Manuals

You can download our User Manuals here. You can also find our User Manuals in other languages.

English

SONESTA S2 & S3 USER MANUAL

SONESTA 6210 USER MANUAL

SONESTA SST USER MANUAL

French

SONESTA S2 & S3 NOTICE D'UTILISATION

SONESTA 6210 NOTICE D'UTILISATION

Spanish

SONESTA S2 & S3 MANUAL DE USUARIO

Quality and Environment

Sonesta is certified according to the international quality standard ISO 14001. Sonesta applies the ADDVise Group Quality and Environment Policy.

Climate change is a real threat to our world. Therefore, we at Sonesta always choose a more environmental friendly way of travelling when we travel, always choosing a more environmental friendly way of travelling when we travel, always choosing a more environmental friendly way of travelling when we travel.

We fully CO2 compensate our travels by airplane with a fee. This fee is paid to a charitable environmental organisation with the power to bring about positive change.

ISO CERTIFICATE

QUALITY AND ENVIRONMENT

CERTIFICATE OF COMPLIANCE

EC DECLARATION OF CONFORMITY - 6210

EC DECLARATION OF CONFORMITY - S2

EC DECLARATION OF CONFORMITY - S3

FCS FREE SALES CERTIFICATE - S2, S3, 6210

