



Unpacking and assembly instructions

Models

S2 & S3

Required tools and capacity for unpacking


One scissor




One powerdrill with a
philips head screw tip



One allen wrench
size no 5. (Included
in the crate)

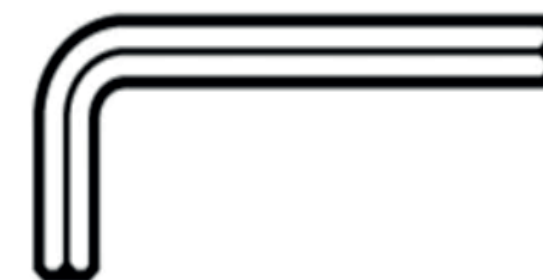

The unpacking includes
moments of heavy lifting
which requires two people

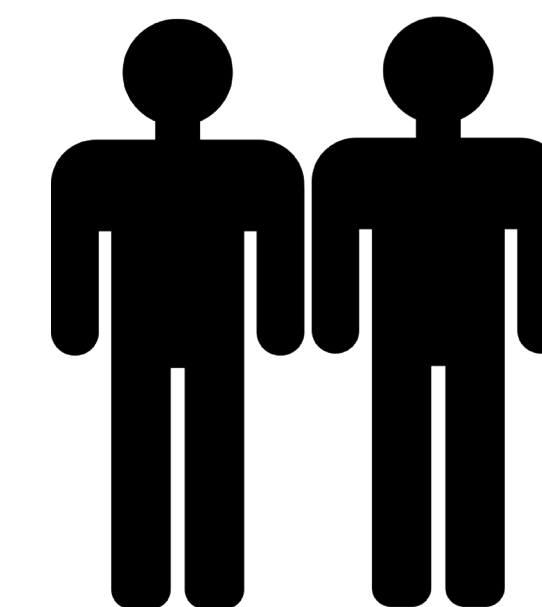

Unpacking and assembly

Step 1: Uncrating the crate

Remove all screws at the base of the crate.

Remove all screws at the lid of the crate.

Cut off the straps surrounding the crate.

Carefully lift off the top of the crate.



Step 2: Remove the plastic

Gently remove the plastic which surrounds and seals the chair, be careful when doing this since there are mattresses beneath the plastic.



Step 3: Unwrap and remove loose articles

The crate comes with articles that are not assembled to the chair, carefully remove these articles and set them aside.

Depending on which model of chair you have ordered the crate will contain different loose articles:

Sonesta S2:

- One Sonesta folder with one allen wrench, no. 5
- Two leg rest holders
- Two leg rests
- One seat cushion
- One funnel and one funnel holder (if ordered)
- One two motor foot control


Sonesta S3:

- One Sonesta folder with one allen wrench, no.5
- Two leg rest holders
- Two leg rests
- Two seat cushions
- One funnel and one funnel holder (if ordered)
- One three motor foot control
- Two footrests
- One neck pillow

Step 4: Install the funnel and funnel holder (if ordered)

To install the funnel holder on the chair slide the funnel holder into the two small brackets located underneath the chair, as shown in the pictures.

Loosen the small black knob on the funnel holder to open the metal ring that holds the funnel. Put the funnel in the metal ring and then tighten the small knob into position.

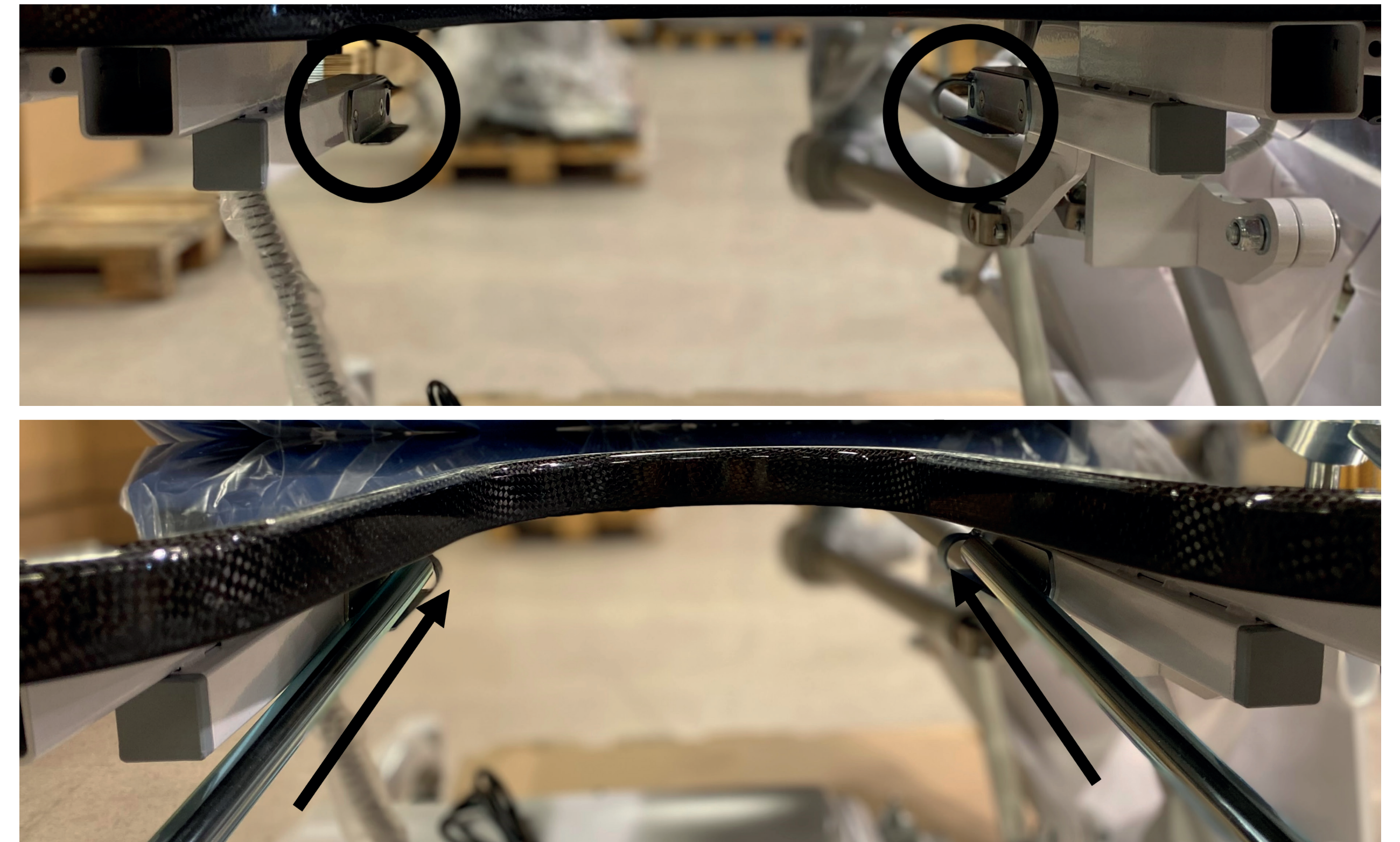

Step 5: Install the leg rest holders

Remove the screw at the end of the metal rail beside the seating with the allen wrench and put the screw aside.

Repeat the step on the other side of the chair.




To install the leg rest holders onto the chair, make sure that the grooves on the holders line up. This will allow for the holder to slide onto the rail.



When the grooves are lined up, slide the holder onto the rail where the screw has been removed. Make sure that the hole in the leg rest holder is pointing upwards.

Screw back the screw at the end of the rail with the allen wrench and tighten.

Repeat the steps on the other side of the chair for the second leg rest holder.

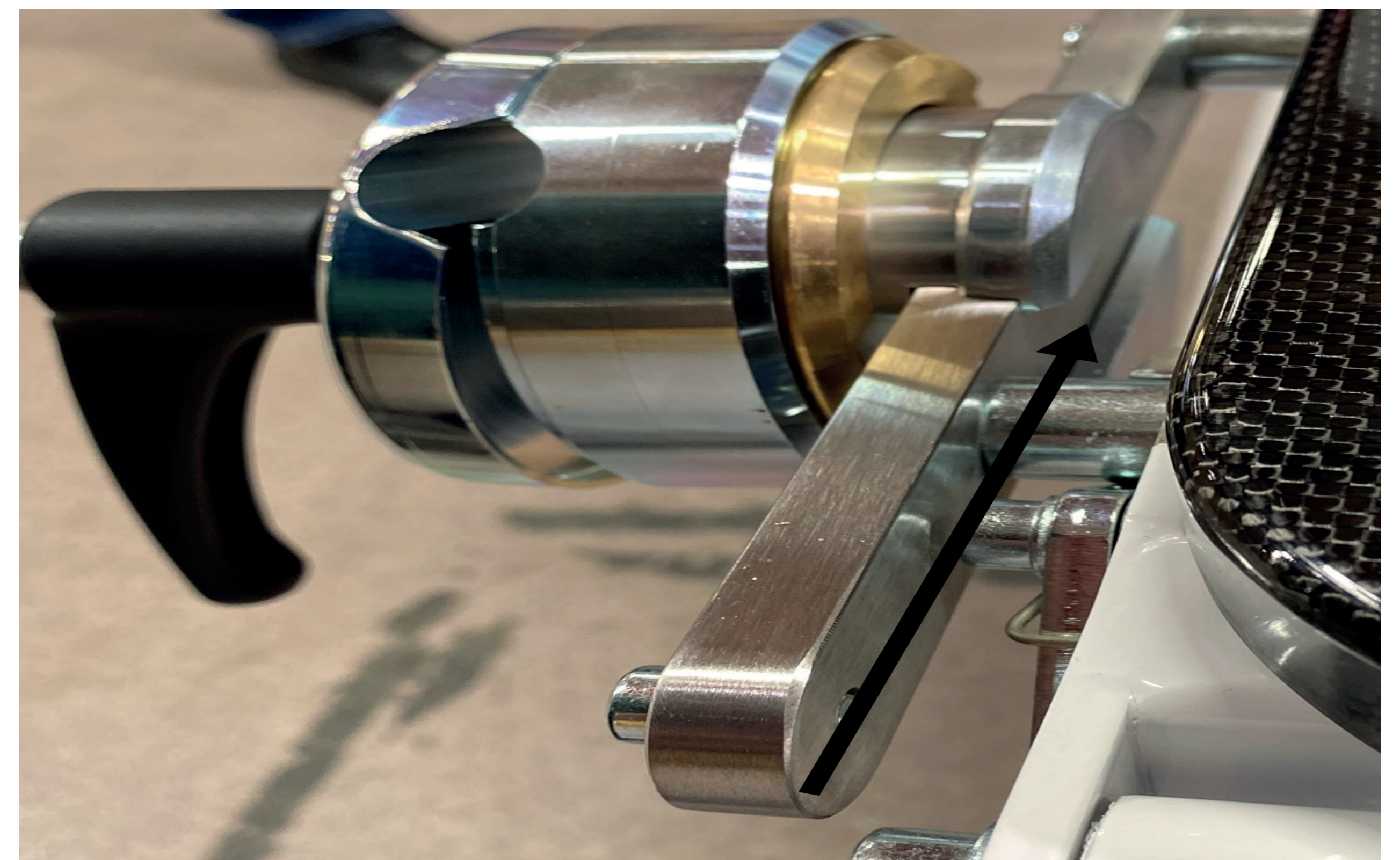

Step 6: Install the leg rests

To install the leg rests, slide the metal pole into the hole on the leg rest holder. Tighten the black handle on the leg rest holders to fasten the leg rest into position.

Repeat the step on the other side of the chair for the second leg rest.




Step 7: Install the footrests (standard accessory for Sonesta S3)

To install the footrest, slide the metal holder at the top of the footrest into the square opening underneath the chair.



Once the black clamp hooks into the metal railing, the footrest is in place. Repeat the step on the other side of the chair for the second footrest.

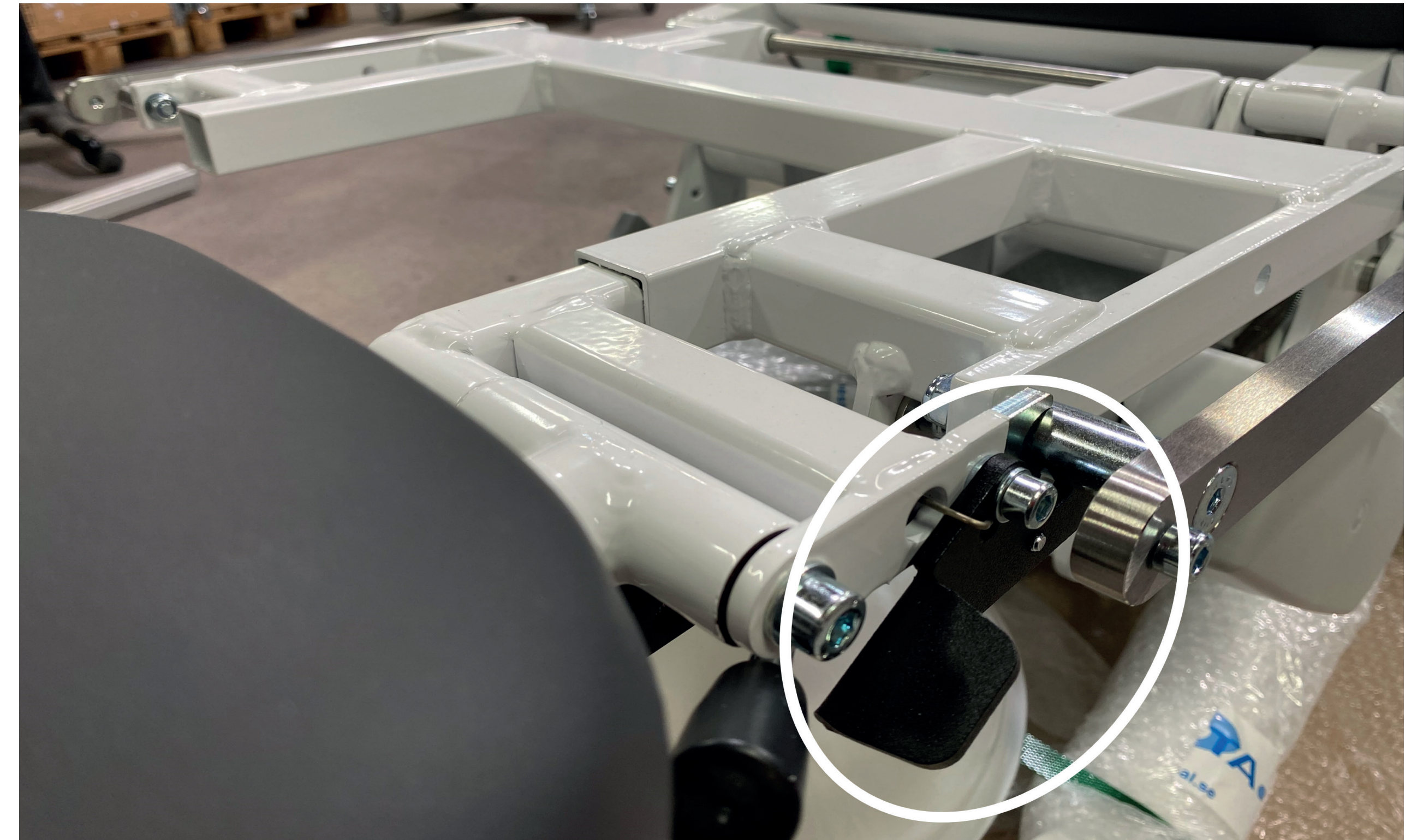

Step 8: Install the seat cushion

To install the seat cushion, slide the seat cushion with the metal bracket facing inwards towards the back seat.

Carefully slide the metal brackets hooks underneath the steel rod at the base of the back cushion. The cushion is in place once the brackets are hooked onto the steel rod.



Step 9: Remove the chair from the pallet

Carefully cut off the straps that hold the chair in place on the pallet.

Unlock the wheels on the chair by lifting the lever towards the inside of the wheel, this allows the chair to freely be moved.

Carefully roll the chair off the pallet.



Notes

Notes

An ADDvise Group Company



SONESTA MEDICAL AB	SONESTA MEDICAL INC
INDUSTRIVÄGEN 7	2 DEBUSH AVENUE UNIT C3
171 48 SOLNA, SWEDEN	MIDDLETON; MA 01949, USA
+46 8 502 572 80	+ 1 630-519-3450
INFO@SONESTAMEDICAL.SE	INFOUS@SONESTAMEDICAL.SE
ORDER@SONESTAMEDICAL.SE	ORDER@SONESTAMEDICAL.SE
WWW.SONESTAMEDICAL.SE	WWW.SONESTAMEDICAL.SE