



Unpacking and assembly instructions

Model

6210

Required tools and capacity for unpacking


One scissor



One powerdrill with a
philips head screw tip



The unpacking includes moments of heavy lifting which requires four people

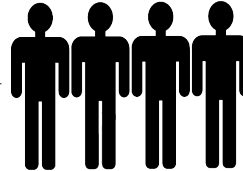

Unpacking and assembly

Step 1: Uncrating the crate

Start by removing the screws at the top of the crate.
Cut off the straps surrounding the crate.

Carefully lift off the top of the crate.
The inside of the lid is reinforced, see picture.



Move on by removing the screws at the side of the crate that is marked out with an attention slip (shown to the right) and lift off the side.

Unscrew the remaining screws on the remaining sides of the pallet and lift of the sides.



ATTENTION
OPEN THIS SIDE OF THE
PALLET FIRST

Step 2: Remove the plastic

Gently remove the plastic which surrounds and seals the table, be careful when doing this since there are mattresses beneath the plastic.



When you have removed the plastic, the accessories will be placed like this:

The fold up foot rests are on top of the mattress. Now you need to mount the foot rests to the table.



Step 3: Mount the fold up foot rests

Mount the foot rests in the marked area.

Make sure you insert the fold up foot rests all the way in



Step 4: Lock the fold up foot rests

Again, make sure you insert the fold up foot rests all the way in and lock it with the locking mechanism.


Unlocked in picture one.

Locked in picture two.

1



2

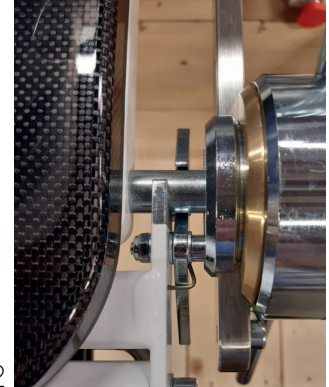

Step 5: Unwrap and remove remaining loose articles

The crate comes with other articles that are not assembled to the table, carefully remove these articles and set them aside.

The crate will now contain the following loose articles:

- One protective cover for the frame
- One arm rest for the right side of the table.


Step 6: Install the right arm rest

Locate the arm rest holder on the right side of the table.

Loosen the holder by turning the black handle counterclockwise.

Press the metal end of the arm rest into the hole on the arm rest holder and tighten the handle into position.

To move the arm rest along the metal railing, loosen the black handle at the bottom of the holder, place the arm rest where you want it and tighten the handle into position.

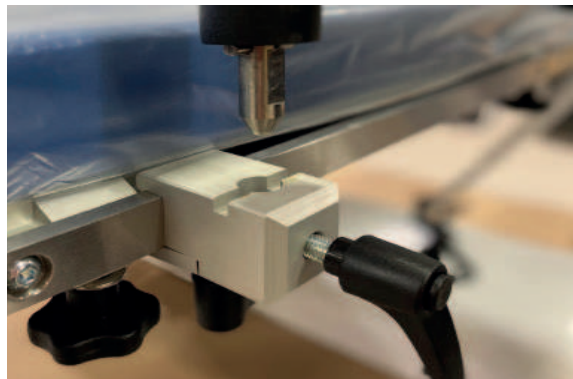

Step 7: Remove the plastic cover on the squeeze protection

Carefully peel off the white protective plastic on the squeeze protection on the left-hand side of the table.



Step 8: Install the protective cover

Gently click the protective cover into position on the left side of the table's base in between the legs.



Step 9: Remove the table from the pallet

Carefully cut off the straps that hold the table in place on the pallet.



Unlock the brakes on the four wheels of the table to allow the wheels to be freely moved once placed on the floor.

To lift the table from the pallet, our suggestion is to have four people help lift the table from the pallet of the frame above the wheels, as shown in the pictures to the right.



ATTENTION!

Do NOT lift the table off the pallet from another location than shown in the pictures to the right.

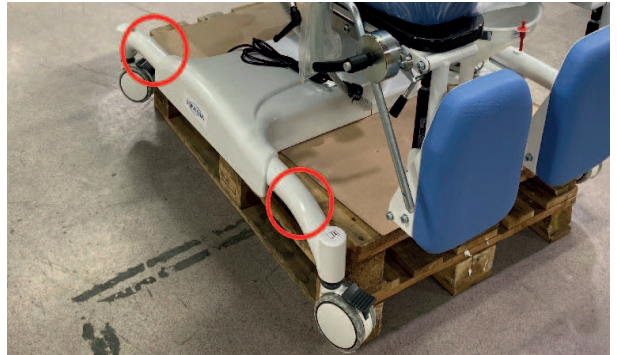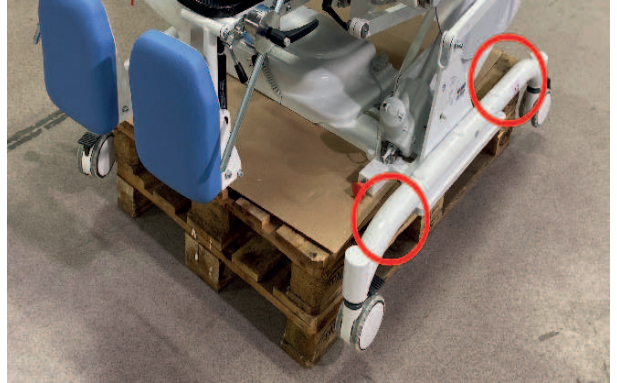

An ADDvise Group Company

QR, Instructions for unpacking



SONESTA MEDICAL AB

INDUSTRIVÄGEN 7

171 48 SOLNA, SWEDEN

+46 8 502 572 80

INFO@SONESTAMEDICAL.SE

ORDER@SONESTAMEDICAL.SE

WWW.SONESTAMEDICAL.SE

SONESTA MEDICAL INC

2 DEBUSH AVENUE UNIT C3

MIDDLETON; MA 01949, USA

+ 1 630-519-3450

INFO@SONESTAMEDICAL.SE

ORDER@SONESTAMEDICAL.SE

WWW.SONESTAMEDICAL.SE