



Cleaning Guide Chairs and accessories

Maintain quality

Regular cleaning is key to preserving the high standards and optimal condition of your Sonesta unit. Make sure that the cleaning guide is followed when cleaning your Sonesta unit.

Safety first

Always disconnect the unit from electrical power before cleaning.

Cleaning steps

Remove fluids immediately, and keep the chair free of blood, urine, and other substances during procedures to prevent them from drying on the surface of the unit.

Clean the unit after each use by wiping down all surfaces and accessories thoroughly.

Keep attention to detail

Seat cushion: Remove the cushion and clean underneath and behind it after each use.

Leg rests and holders: Detach from the side rail for comprehensive cleaning.

Cleaning solutions

Clean the unit using a **damp cloth** with one of the following listed disinfectants:



- PDI Sani-Cloth® Bleach Wipe
- Clorox HealthCare® Disinfecting Wipes
- Ytidesinfektion LIV Des +45®

If the listed disinfectants are not available, use any disinfectant with a **1:10 bleach dilution** or **45% isopropyl alcohol**.

Warning!

Do not use Anios Surfa'Safe Premium Foaming Spray on the unit. It does not contain enough alcohol to be effective for disinfection.

Take care

Avoid damage: Inadequate cleaning of the unit can cause corrosion, damage the appearance, and affect the performance.

Beware of stains: Iodophor disinfectants (such as betadine) may stain the cushions.

Proper dilution is crucial: Incorrect bleach concentration can harm the functionality and appearance of the unit.