



Unpacking and assembly instructions

Model

F3

Required tools and capacity for unpacking


One scissor



One powerdrill with a
philips head screw tip



The unpacking includes moments of heavy lifting which requires four people

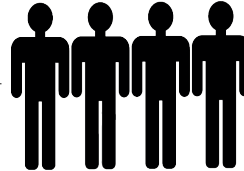

Unpacking and assembly

Step 1: Uncrating the crate

Start by removing the screws at the top of the crate.
Cut off the straps surrounding the crate.

Carefully lift off the top of the crate.
The inside of the lid is reinforced, see picture.



ATTENTION
OPEN THIS SIDE OF THE
PALLET FIRST

Move on by removing the screws at the side of the crate that is marked out with an attention slip (shown to the right) and lift off the side.

Unscrew the remaining screws on the remaining sides of the pallet and lift of the sides.

Step 2: Remove the plastic

Gently remove the plastic which surrounds and seals the table, be careful when doing this since there are mattresses beneath the plastic.



When you have removed the plastic, the accessories will be placed like this:

The fold up foot rests are on top of the mattress. Now you need to mount the foot rests to the table.



Step 3: Mount the fold up foot rests

Mount the foot rests in the marked area.

Make sure you insert the fold up foot rests all the way in



Step 4: Lock the fold up foot rests

Again, make sure you insert the fold up foot rests all the way in and lock it with the locking mechanism.


Unlocked in picture one.

Locked in picture two.

1



2

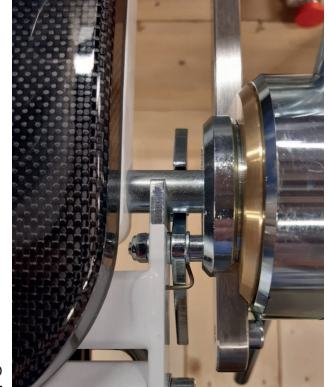

Step 5: Remove the plastic cover on the squeeze protection

Carefully peel off the white protective plastic on the squeeze protection on the left-hand side of the table.



Step 6: Remove the table from the pallet

Carefully cut off the straps that hold the table in place on the pallet.

Unscrew the 5 boards that anchor the base. There are 2 long ones with 4 screws on each and 3 short ones with 2-3 screws on each. See the white arrows in the picture.


Unlock the brakes on the four wheels of the table to allow the wheels to be freely moved once placed on the floor.

To lift the table from the pallet, our suggestion is to have four people help lift the table from the pallet of the frame above the wheels, as shown in the pictures to the right.



ATTENTION!

Do NOT lift the table off the pallet from another location than shown in the pictures to the right. See red circles.

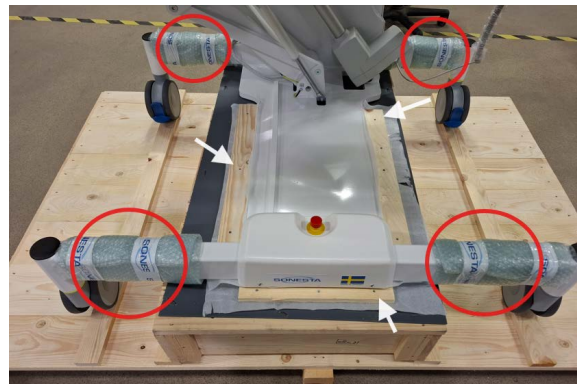

An ADDvise Group Company

QR, Instructions for unpacking



SONESTA MEDICAL AB

INDUSTRIVÄGEN 7

171 48 SOLNA, SWEDEN

+46 8 502 572 80

INFO@SONESTAMEDICAL.SE

ORDER@SONESTAMEDICAL.SE

WWW.SONESTAMEDICAL.SE

SONESTA MEDICAL INC

2 DEBUSH AVENUE UNIT C3

MIDDLETON; MA 01949, USA

+ 1 630-519-3450

INFO@SONESTAMEDICAL.SE

ORDER@SONESTAMEDICAL.SE

WWW.SONESTAMEDICAL.SE