



# Sonesta 6210 & F3 Seat Cover

## HOW TO APPLY COVER

1. Remove one seat cover from the package.
2. Position the cover at the front of the seat, aligning the central opening of the cover with the cut-out of the cushion.
3. Pull the cover forward and downward so that both the mattress surface and the exposed front carbon fiber edges are fully covered.
4. Continue pulling the cover rearward over the seat so that it fully seats against the back of the cushion.
5. Smooth the cover to remove folds and ensure it lies flat without tension.
6. Verify that the mattress is correctly positioned on the carbon fiber base and that no structural surfaces are exposed.
7. After use on one patient, remove and discard the cover.



Step 1



Step 2

## DEVICE DESCRIPTION

The Sonesta 6210 & F3 Seat Cover is a disposable, single-use cover intended to provide a protective barrier between the patient and the examination chair.

**Article number:** 722505

**Quantity:** 20 pcs / package

## INTENDED USE

- For use on Sonesta 6210 and Sonesta F3 examination chairs.
- Applied before patient contact and removed and discarded after one patient.

## WARNING AND PRECAUTIONS

- Single patient use only.
- Do not reuse.
- Do not use if the package is damaged.
- Non-sterile.
- Ensure the cover is correctly positioned before patient use.
- Do not use the cover as a wound dressing, surgical drape, or protective barrier for other purposes.
- Not intended for contact with broken skin, wounds, or mucous membranes.

## DISPOSAL

Discard the used seat cover according to local waste handling procedures.

## STORAGE

Store in a clean and dry environment. Protect from moisture and damage.



### Manufacturer

Sonesta Medical AB  
Industrivägen 7  
SE-171 48 Solna  
Sweden

