



Procedure and Examination Chairs

ACCESSORY CATALOG



Models

| P2 | P3 |

Sonesta manufactures and sells tables and chairs for urology, gynecology, urodynamic diagnostics and upper GI procedures.

Accessories: P2 & P3

Leg & foot rests

P2 P3 Art. number

Leg rest holder (single)

Allows the leg rest to be adjusted in height, width and angle. The holder can be rotated 360 degrees and repositioned as needed.



A A

525-6338

Leg rests (pair)

The ball-and-socket design allows the leg cushion to tilt in any direction. Used with the leg rest holder, the stainless-steel leg rests provide flexible patient positioning.



A A

525-6336

Fold-up footrests (pair)

These footrests are adjustable in length from 30—42 cm/11,8"—16,5". They are also adjustable in width and slide outward 19 cm/7,5" each. The bottom is in the same upholstery as your chair. When folded they function as an extension cushion that transforms your chair into a table where the patient can lie down. The gas piston easily adjusts the angle. They are easily removed with a click-off system.



A A

527-120564

Footplates (pair)

Made of stainless steel with a plastic foot section, fits into our standard leg rest holder.



A A

524-P6413

Stirrups and clamps (pair)

Stirrups that provide safe and quick positioning which improves surgical site access. The stirrups comes with silicone straps for easier cleaning and attachment. Lithotomy range: +85° to +55°, abduction range: +25° to -9°, boot rotational angle: +/- 25°.



A A

525-3100

Patient weight capacity: 350 lbs / 159 kg

Benefits & use:

- Multiple positioning options
- Patient comfort
- Enables optimal examination access
- Engineered ergonomics

S

Standard configuration

A

Optional accessory

Seat cushions

P2

P3

Art. number

Seat cushion with 170 mm cutout

A 'quick change' accessory seat cushion, with a 170 mm/6,7" deep cutout, that can be switched in seconds with any other seat cushion. You simply lift the front edge, remove it and replace with another type of cushion.



S

S

527-120565

Seat cushion with 220 mm cutout

A 'quick change' accessory seat cushion, with a 220 mm/8,7" deep cutout, that can be switched in seconds with any other seat cushion. You simply lift the front edge, remove it and replace with another type of cushion.



A

A

527-120566

Seat cushion with 80 mm cutout

A 'quick change' accessory seat cushion for pediatric care, with a 80 mm/3,1" deep cutout, that can be switched in seconds with any other seat cushion. You simply lift the front edge, remove it and replace with another type of cushion.



A

A

527-120567

Seat cushion without cutout

A 'quick change' accessory seat cushion that can be switched in seconds with any of our other seat cushions. You simply lift the front edge, remove it and replace with another type of cushion.



A

A

527-120568

Benefits & use:

- Multiple seating options
- Patient comfort
- Enables optimal examination access
- Easy to use and interchangeable
- Ergonomic and low profile cushions

S

Standard configuration

A

Optional accessory

Extension & neck cushions

P2

P3

Art. number

Extension cushion

"Slide-in" extension cushion fits into the two square pipes under the front of the seat cushion and is easily removed when the patient's legs are put onto the leg rests. Makes the chair into a 182 cm/72" long flat examination table where your patient can lie down comfortably.



A

A

527-120195

Puzzle piece extension cushion

Puzzle piece "slide-in" extension cushion that fills the standard 170 mm cushion cutout. Fits into the two square pipes under the front of the seat cushion and is easily removed when patient's legs are put onto the leg rests. Makes the chair into a 193 cm/76" long flat examination table where your patient can lie down comfortably.



A

A

527-120569

Neck pillow

An adjustable neck pillow in the same upholstery as the chair. Its position can be adjusted to suit the patient's height.



A

A

527-120571

Benefits & use:

- Multiple positioning options
- Ability to transform your chair into a flat bed
- Patient comfort
- Adjustable in length, 134 cm—182 cm/53"—72"
- User-friendly
- Ergonomic and low profile cushions

S

Standard configuration

A

Optional accessory

Accessories: P2 & P3

Funnels

P2

P3

Art. number

Funnel kit

A removable plastic funnel (top diameter 236 mm/9") with drainage. It is mounted on a fixed height 'quick change' holder.



A

A

524-6325-5

Drainage receptacle

Standard receptacle 60 mm/2,4" deep – A stainless steel receptacle with an adjustable 'quick change' holder.



A

A

524-6325-1

Funnel and cysto pan kit

The Cysto pan comes standard with a tray that mounts to the chair that holds the pan in place. The pan comes equipped with a screen, a plug and a hose that extends to 193 cm/76".



A

A

524-6325-10

Benefits & use:

- Practical use
- Functional design
- Easy to use and interchangeable

S

Standard configuration

A

Optional accessory

Controls

Two-motor foot control

Waterproof foot control for Sonesta P2 chairs.



P2 P3

Art. number

A -

527-120574

Three-motor foot control

Waterproof foot control for Sonesta P3 chairs.



- A

527-120576

Two-motor hand control

Waterproof hand control for Sonesta P2 chairs.



S -

527-120572

Three-motor hand control

Waterproof hand control for Sonesta P3 chairs.



- S

527-120573

Benefits & use:

- Engineered ergonomics
- User-friendly
- Precise positioning

S

Standard configuration

A

Optional accessory

Other accessories



Art. number

IV pole

An IV pole with holder that attaches to a support under the seat cushion. The IV pole will stay in the height relative to the patient as it moves as the table is adjusted. To be mounted on patient's right side of the chair.



524-6316

Arm rest (single)

The arm rest functions as both an arm support and a handgrip. It can be rotated away to facilitate access and patient transfer.



527-120579 (L)
527-120578 (R)

Backup battery


Backup battery for Sonesta P2 and P3 chairs, providing temporary chair operation when mains power is unavailable.



527-120577

Restraint strap

Velcro straps for patient fixation. Pair of 100 cm/39" long and 10 cm/3,9" wide straps.

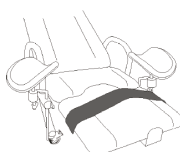

524-P6415

Benefits & use:

- Versatility
- Helpful examination tools
- Easy access



Standard configuration



Optional accessory

How to accessorize

How to accessorize to meet your and your patient's needs



Arm rests

Easy access to the patient from each side.



Leg rests

Swivelling leg rests for ultimate patient positioning and comfort.



Foot rests

Gas piston helps switch from foot rests to extension cushion.



Foot plates

Convenient foot plates for function and comfort.



Neck pillow

An ergonomic accessory that enhances the patient comfort.



Puzzle piece

An extension cushion that transforms your chair into a table or flat bed.



Seat cushion

Choose the perfect fit for your patient and examination.



Motor control

Ergonomic motor controls enables precise positioning.



Funnel

A functional funnel kit for performing non-UDS urology procedures.



Backup battery

Backup power for temporary chair operation.



Drainage receptacle

A stainless steel receptacle. Perfect for your instruments.



IV pole

Allows the patient to remain in one place during procedures.



SONESTA MEDICAL AB
INDUSTRIVÄGEN 7
171 48 SOLNA, SWEDEN
+46 8 502 572 80
INFO@SONESTAMEDICAL.SE
ORDER@SONESTAMEDICAL.SE
WWW.SONESTAMEDICAL.SE

SONESTA MEDICAL INC.
2 DEBUSH AVENUE UNIT C3
MIDDLETON, MA 01949, USA
+1 630-519-3450
INFO@SONESTAMEDICAL.SE
ORDER@SONESTAMEDICAL.SE
WWW.SONESTAMEDICAL.SE